# FISIO CHAMBER

vision

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KM-1025

KOO (Shanghai) Industries Co., Ltd.  
100 Zhongde Road, Dakun Industrial Park  
Songjiang Shanghai, CHINA

Εργοστάσιο Παραγωγής  
KOO Medical Equipment (Shanghai) Co., Ltd.  
1 Songjiang Industry Park Shanghai, CHINA

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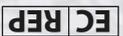
Koo Medical Equipment (Shanghai)  
Co., Ltd  
100 Zhongde Road, Dakun Industrial  
Park, Songjiang Shanghai, 201614,  
P.R. CHINA

KM-1020  
STANDARD  
STANDARD  
STANDARD  
ERWACHSENE  
STANDARD  
KΑΝΟΝΙΚΟΣ  
STANDARD  
ESTANDARD

BPA FREE

vision

FISIO CHAMBER

- ANTI-STATIC VALVED INHALER CHAMBER FOR METERED DOSE INHALER
- CAMERA DI INALAZIONE ANTI-STATICA CON VALVOLA PER INALATORI SPRAY PREDOSATI
- ANTISTATISCHE INHALIERKAMMER MIT VENTIL FÜR DOSIERAEROSOL
- ANTISTATISCHE INHALATIEKAMER MET VENTIEL VOOR SPRAYINHALATIEOESTELLEN MET DOSEERMETING
- ΣΕΤ ΘΑΛΑΜΟΥ ΧΟΡΗΓΗΣΗΣ ΕΙΣΠΝΕΟΜΕΝΩΝ ΦΑΡΜΑΚΩΝ ΜΕ ΑΝΤΙΣΤΑΤΙΚΗ ΒΑΛΒΙΔΑ ΓΙΑ ΧΡΗΣΗ ΜΕ ΔΟΣΟΜΕΤΡΙΚΗ ΣΥΣΚΕΥΗ ΕΙΣΠΝΟΗΣ
- CAMARA INHALADORA ANTIESTATICA PARA INHALADOR DOSIFICADOR
- DOZ AYARLI SOLUNUM İÇİN ANTI-STATİK VALFLİ INHALER HAZNESİ
- CHAMBRE D'INHALATION ANTISTATIQUE POUR AÉROSOL DOSEUR

FISIO CHAMBER vision

BPA FREE

KM-1020

STANDARD  
STANDARD  
ERWACHSENE  
STANDAARD  
KΑΝΟΝΙΚΟΣ  
ESTANDARD  
STANDARD  
STANDARD

- ANTI-STATIC VALVED INHALER CHAMBER FOR METERED DOSE INHALER
- CAMERA DI INALAZIONE ANTI-STATICA CON VALVOLA PER INALATORI SPRAY PREDOSATI
- ANTISTATISCHE INHALIERKAMMER MIT VENTIL FÜR DOSIERAEROSOL
- ANTISTATISCHE INHALATIEKAMER MET VENTIEL VOOR SPRAYINHALATIEOESTELLEN MET DOSEERMETING
- ΣΕΤ ΘΑΛΑΜΟΥ ΧΟΡΗΓΗΣΗΣ ΕΙΣΠΝΕΟΜΕΝΩΝ ΦΑΡΜΑΚΩΝ ΜΕ ΑΝΤΙΣΤΑΤΙΚΗ ΒΑΛΒΙΔΑ ΓΙΑ ΧΡΗΣΗ ΜΕ ΔΟΣΟΜΕΤΡΙΚΗ ΣΥΣΚΕΥΗ ΕΙΣΠΝΟΗΣ
- CAMARA INHALADORA ANTIESTATICA PARA INHALADOR DOSIFICADOR
- DOZ AYARLI SOLUNUM İÇİN ANTI-STATİK VALFLİ INHALER HAZNESİ
- CHAMBRE D'INHALATION ANTISTATIQUE POUR AÉROSOL DOSEUR

LOT

MADE IN CHINA

LPP 1128423

 Inhalation Vision Valve • Universal Adapter “Proper Fit” • Spiral Vents

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In order to keep the full efficiency of the device, we advise to change the valve block every 12 months (ref. KM-1028)

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
## instructions for use


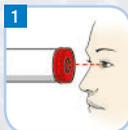
**1** Before use, carefully check all the parts, valves and holes.

Replace if any defect is noticed.

Remove cap from MDI (Metered Dose Inhaler).

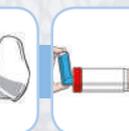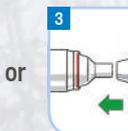Shake the MDI per instructions. Insert and seal the MDI into the universal adapter "Proper Fit".

OR

Remove cap from Chamber.

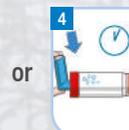
Apply the mouthpiece into the mouth and verify to have properly sealed.

OR

According to the patient's needs, you can insert the Fisio Mask (NOT INCLUDED) into mouthpiece of the chamber. Verify the mask safety valve is openable (if not, replace it). Apply the mask to the face. Verify to have properly sealed.

vision valve

OR

Inhale and exhale normally.

Then press the MDI one time and start to inhale slowly.

Carefully look at the VISION VALVE as the movement shows the inhalation act.

Maintain seal the mouthpiece (or the mask, if used) and breath for at least 5 inhalation acts after MDI is pressed. After every inhale, hold your breath for at least 5 seconds to allow the medicine deposit in the lungs.

Wait before a new puff and repeat this step as for doctor or MDI instructions. Always follow prescribed dosage.

## cleaning instructions

Clean before first use, then at least once a week following instructions.

**1** Remove parts as shown and open the chamber.

**2** Soak the parts for 15 minutes in lukewarm clean water and mild detergent.

Agitate gently.

Rinse the parts in clean water.

Shake out excess of water and allow to air dry.

Do not use dry clothes to dry. Make sure all the parts are dry before reassembly. Reassembly parts as shown and stock the product in a dry and clean container.

Storage Temperature -10°C / +45°C

Relative Humidity 30% / 80%

### CAUTIONS

Follow instructions before every use. Do not leave Fisio Chamber Vision or part of it unattended with children, as some components are small enough to be swallowed. This product is a Single Patient Use.

Do not use with different patients to prevent contaminations. The product should be replaced after max. 2 years of continual use. Replace the product if damage, sealing of the valve, environmental condition, storage and frequency of use make the product different from the original use. Dispose according regulations.

The use and the sale of Fisio Chamber Vision is restricted by or on the order of physician. Monitor the patient during use.

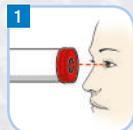
A Anti-static chamber

B Inhalation Vision Valve

C Universal adapter "Proper Fit" with spiral vents

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
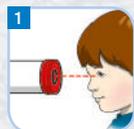
## instructions for use

**1** Before use, carefully check all the parts, valves and holes.  
Replace if any defect is noticed.

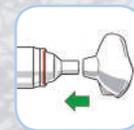**2** Remove cap from MDI (Metered Dose Inhaler).  
Shake the MDI per instructions.  
Insert and seal the MDI into the universal adapter "Proper Fit".

or

**3** Remove cap from Chamber.  
Insert the Fisio Mask into mouthpiece of the chamber.  
Verify the mask safety valve is openable (if not replace it).  
Apply the mask to the face. Verify to have properly sealed.

OR

If the patient can cooperate, apply the mouthpiece only into the mouth and verify to have properly sealed.

**4** Inhale and exhale normally.  
Then press the MDI one time and start to inhale slowly.  
Carefully look at the VISION VALVE as the movement shows the inhalation act.

Maintain seal the mask or the mouthpiece for at least 5 inhalation acts after MDI is pressed.

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vision valve



### CAUTIONS

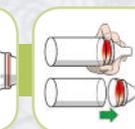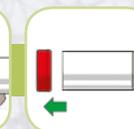Follow instructions before every use. Do not leave Fesio Chamber Vision or part of it unattended with children, as some components are small enough to be swallowed. This product is a Single Patient Use. Do not use with different patients to prevent contaminations. The product should be replaced after max. 2 years of continual use. Replace the product if damage, sealing of the valve, environmental condition, storage and frequency of use make the product different from the original use. Dispose according regulations.

The use and the sale of Fesio Chamber Vision is restricted by or on the order of physician. Monitor the patient during use.

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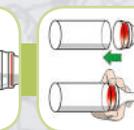
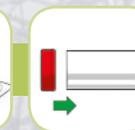
**2** Soak the parts for 15 minutes in lukewarm clean water and mild detergent.  
Agitate gently.

**3** Rinse the parts in clean water.  
Shake out excess of water and allow to air dry.

Do not use dry clothes to dry.  
Make sure all the parts are dry before reassembly.

Reassembly parts as shown and stock the product in a dry and clean container.


Storage Temperature  $-10^{\circ}$  /  $+45^{\circ}$ C  
Relative Humidity 30% / 80%

- A Anti-static chamber
- B Inhalation Vision Valve
- C Universal adapter "Proper Fit" with spiral vents
- D Exhalation port (DO NOT OCCLUDE)
- E Mask Comfort & Seal with exhalation safety valve

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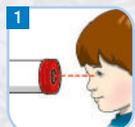**3** Remove cap from Chamber.  
Insert the Fiso Mask into mouthpiece of the chamber.  
Verify the mask safety valve is openable (if not replace it).  
Apply the mask to the face.  
Verify to have properly sealed.

**4** Inhale and exhale normally. Then press the MDI one time and start to inhale slowly.  
Carefully look at the VISION VALVE as the movement shows the inhalation act.  
Maintain seal the mask for at least 5 inhalation acts after MDI is pressed.  
Wait before a new puff and repeat this step as for doctor or MDI instructions.  
Always follow prescribed dosage.

vision valve


### CAUTIONS

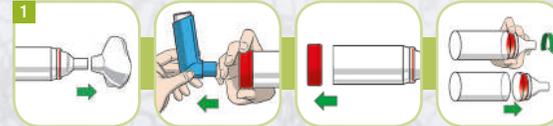Follow instructions before every use. Do not leave Fiso Chamber Vision or part of it unattended with children, as some components are small enough to be swallowed. This product is a Single Patient Use. Do not use with different patients to prevent contaminations. The product should be replaced after max. 2 years of continual use. Replace the product if damage, sealing of the valve, environmental condition, storage and frequency of use make the product different from the original use. Dispose according regulations.

The use and the sale of Fiso Chamber Vision is restricted by or on the order of physician. Monitor the patient during use.

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
Remove parts as shown and open the chamber.

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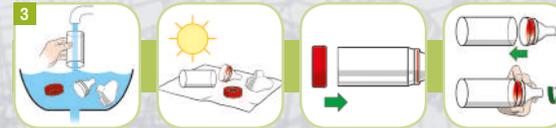**3** Rinse the parts in clean water.  
Shake out excess of water and allow to air dry.  
Do not use dry clothes to dry.  
Make sure all the parts are dry before reassembly.

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Storage Temperature -10° / +45°C  
Relative Humidity 30% / 80%

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